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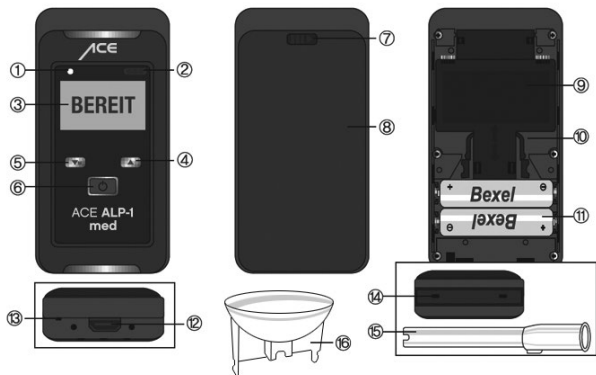
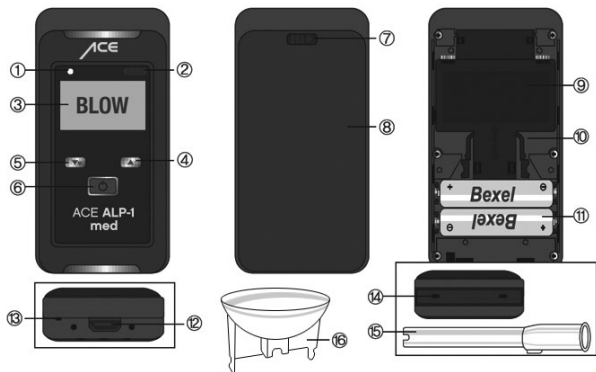
ACE ALP-1 med Breathalyzer Quick Start Guide

(please read the manual for further details)

ACE
Instruments
PRECISION OR NOTHING.

Label of individual components:

1. LED light
2. IrDA window
3. Digital LC display
4. Up button
5. Menu / down button
6. Power / select button
7. Button for opening battery cover
8. Battery cover

9. Sensor module
10. Handle of sensor module
11. Batteries
12. USB connector
13. Connection for hand strap
14. Mouthpiece receptacle
15. Mouthpiece
16. Breath funnel (used for tests in fast mode, see the details about fast mode in the manual)

Conducting the measurement:

1. Shortly press the power button on the front of the device.
2. The device warms up until you can see the "BLOW" sign and hear one beep sound. During the display WAIT sign you can go into menu mode if you press the button "M". If you leave the menu mode, warming up is automatically done.
3. When "BLOW" is shown on the display also a green light is blinking above the display, this green light blinks until you start your test.
4. Now constantly blow into the mouthpiece for 3 seconds until a "tictac" tone concludes the breath sample. Tip: Blow into the mouthpiece as if you would whistle without a tone.
5. After taking your breath sample the LC display shows "Analyzing/wait".
6. The result is displayed with one beep sound, you can see the result for a time period of 15 seconds.
7. The device automatically turns off after 15 seconds without using it. If you want to do another test, then shortly press the power button again. "If you keep press and hold the power switch for 3 seconds, you can see "Power off" on display. After showing this message for 2 seconds the device turns off.

Please note:

- Wait at least 15 minutes after finishing your last drink.
- From hygiene reasons and for accurate measurement results always use a new mouthpiece for each test.
- The default measurement setting for EU is per mill, for UK it is mg/L and for the USA it is % BAC.