# Dräger DrugCheck 3000

## Instructions for use

de, enUS, fr, es, ptBR

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en	US	Instructions for use
1	1	<b>Safety-related information</b>
1.1	1.1	<b>Basic safety instructions</b>
–	–	Comply with all local and national rules and regulations associated with this product.
–	–	Notify Dräger in the event of any component fault or failure.
–	–	Do not use a faulty or incomplete product. Do not modify the product.
–	–	Only trained and competent users are permitted to use this product.
–	–	Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
–	–	Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.
1.2	1.2	<b>Precautions and warnings</b>
1.2.1	1.2.1	<b>Ensuring proper hygiene</b>
–	–	Follow proper handling and disposal procedures.
–	–	Open the foil pouch only immediately before use to prevent contamination of the sample collector.
–	–	Do not use the Dräger DrugCheck 3000 if the foil pouch is damaged in any way.
–	–	Do not use the Dräger DrugCheck 3000 beyond the expiration date indicated on the foil pouch. The expiry date is in the format YYYY-MM. Example: 2024-01 means that the Dräger DrugCheck 3000 is not permitted to be used after the end of January 2024.
1.2.2	1.2.2	<b>Product overview</b>
1.2.2.1	1.2.2.1	<b>Dräger DrugCheck 3000 Test Kit (Fig. A)</b>
1	1	Sample collector with color indicator
2	2	Test cassette with indicator ring
3	3	Funnel-shaped test opening
4	4	Security seal
5	5	Result window with test strips
6	6	Control lines area (C)
7	7	Test lines area (MET, AMP, THC, BZO, OPI, COC)
		Each packaged unit contains:
–	–	20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
–	–	1 x Instructions for Use
1.2.2.2	1.2.2.2	<b>Result window with test strips (Fig. B)</b>
6	6	Control lines area (C)
7	7	Test lines area (MET, AMP, THC, BZO, OPI, COC)
		Each packaged unit contains:
–	–	20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
–	–	1 x Instructions for Use
1.2.2.3	1.2.2.3	<b>Product overview</b>
1.2.2.3.1	1.2.2.3.1	<b>Dräger DrugCheck 3000 Test Kit (Fig. A)</b>
1	1	Sample collector with color indicator
2	2	Test cassette with indicator ring
3	3	Funnel-shaped test opening
4	4	Security seal
5	5	Result window with test strips
6	6	Control lines area (C)
7	7	Test lines area (MET, AMP, THC, BZO, OPI, COC)
		Each packaged unit contains:
–	–	20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
–	–	1 x Instructions for Use
1.2.2.3.2	1.2.2.3.2	<b>Result window with test strips (Fig. B)</b>
6	6	Control lines area (C)
7	7	Test lines area (MET, AMP, THC, BZO, OPI, COC)
		Each packaged unit contains:
–	–	20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
–	–	1 x Instructions for Use

**1 Safety-related information**

**1.1 Basic safety instructions**

- Comply with all local and national rules and regulations associated with this product.
- Notify Dräger in the event of any component fault or failure.
- Do not use a faulty or incomplete product. Do not modify the product.
- Only trained and competent users are permitted to use this product.
- Do not dispose of the instructions for use. Ensure that they are stored and used appropriately by the user.
- Strictly follow the instructions for use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the intended use section of this document.

**1.2 Precautions and warnings**

**1.2.1 Ensuring proper hygiene**

- Follow proper handling and disposal procedures.
- Open the foil pouch only immediately before use to prevent contamination of the sample collector.
- Do not use the Dräger DrugCheck 3000 if the foil pouch is damaged in any way.
- Do not use the Dräger DrugCheck 3000 beyond the expiration date indicated on the foil pouch. The expiry date is in the format YYYY-MM. Example: 2024-01 means that the Dräger DrugCheck 3000 is not permitted to be used after the end of January 2024.

**1.2.2 Product overview**

**1.2.2.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**1.2.2.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**1.2.2.3 Product overview**

**1.2.2.3.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**1.2.2.3.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.2 Description**

**2.1 Product overview**

**2.1.1 Dräger DrugCheck 3000 Test Kit (Fig. A)**

- Sample collector with color indicator
- Test cassette with indicator ring
- Funnel-shaped test opening
- Security seal
- Result window with test strips
- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**2.1.2 Result window with test strips (Fig. B)**

- Control lines area (C)
- Test lines area (MET, AMP, THC, BZO, OPI, COC)

Each packaged unit contains:

- 20 Dräger DrugCheck 3000 test kits, each individually packaged in a sealed foil pouch
- 1 x Instructions for Use

**Symbol Explanation**

- Intended for single use only
- Batch
- Order number
- Temperature range for storage conditions
- Expiration date
- Contents sufficient for <math>\leq 60</math> tests
- Measurement mode: fast (THC)
- Measurement mode: sensitive (THC)

**2.4 Test principle**  
The Dräger DrugCheck 3000 is based on the immunoassay principle of competitive inhibition. Drugs contained in the sample compete with drugs on the test membrane for the cohesion of nanoparticles coated with antibodies.

**Sampling:**  
The Dräger DrugCheck 3000 is designed for use with oral fluid samples, which are taken with the integrated sample collector.

The sample does not require special

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© Dräger Safety AG & Co. KGAA