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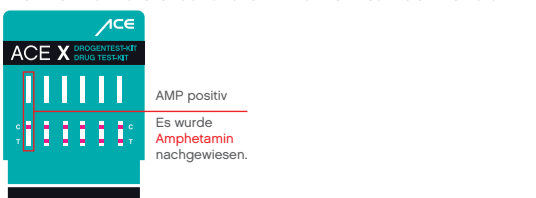
Instruction manual – EN
RAPID DRUG TEST for the detection of amphetamine, barbiturates, benzodiazepines, ketamine, cocaine, methadone, methamphetamine, opiates, spice, tricyclic antidepressants, marijuana and ecstasy-MDMA

DRUG TEST KIT ACE X12, ACE Art. No.: 100340

INTENDED USE: You can use the ACE X12 rapid drug test safely, easily, quickly and anywhere. In the instructions you will learn how to detect a wide variety of substances using a modern test procedure. This test method is also used by the police, among others, for testing.

WITH ACE X12 RAPID DRUG TEST YOU CAN... Urine samples that can be used to detect the use of drugs ...SURVEY.

PACKAGE CONTENTS

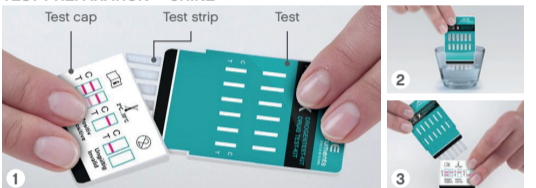
- 1 instruction manual
- 5 shrink-wrapped test cassettes

You will also need a watch, disposable gloves and a test cup. **IMPORTANT INFORMATION:** Everyone is confronted with legal drugs such as alcohol and cigarettes in everyday life. Unfortunately, young people in particular are increasingly experiencing illegal substances such as narcotics. And what begins with curious experimentation often ends in addiction or dependency. Everyone is at risk of becoming addicted. Many people cite insecurity, a lack of conflict resolution, peer pressure, fitting in with friends or escaping from everyday problems as the causes. The boundaries between experimentation and addiction are fluid. The identifying features can be very diverse. There are indications that point to regular drug use. These include, for example, a change in character. People who take drugs can become aggressive or apathetic. Their school performance often suffers and it is not uncommon for their ability to work to be impaired. A conspicuous lack of interest in the environment or carelessness about one's own appearance can also be a sign of drug use. In the event of suspicion, the ACE X12 rapid drug test provides certainty as to whether drug use may actually be the cause. No one can watch their child or partner 24 hours a day to detect or prevent drug use. Such overprotection sometimes even has the opposite effect of what you want, and the other person feels monitored. It is much more important to give children protective mechanisms for life. The existing relationship of trust with your partner should be further developed and strengthened. Ideally, you should talk to them and then take the test together. The rapid test can provide demonstrable clarity as to whether your child or partner is actually using drugs. The ACE X12 rapid drug test can also be used as a self-test, for example before a job interview. If necessary, the test can be repeated at any time to provide evidence of consistent abstinence from drugs.

HOW CAN THE ACE X12 RAPID DRUG TEST HELP? Parents or partners of addicts can also get further advice and help from addiction counseling centers via the Federal Center for Health Education or the Federal Association of Pediatricians and Adolescent Doctors.

DETECTION OF DRUGS IN URINE: Instructions for using the ACE X12 rapid drug test to detect drugs in urine samples. Drugs can be detected beyond doubt in a urine sample. It is therefore the exact detection material for the ACE X12 rapid drug test. To prevent tampering with a urine sample, please ensure that you do not cheat when submitting the sample. No other liquids may be added to the test cup. Contamination with cleaning agents, which can falsify the test result, must be prevented. Everyone involved in the test should wear gloves to avoid contact with the urine sample.

TEST PREPARATION – URINE

1. Remove the foil cover from the test cassette (foil and desiccant can be disposed of with household waste). Open the cap of the test cassette. The sample collection area is free. Place the cassette on a dry, clean surface.
2. Dip the test strips vertically into the urine for at least 10-15 seconds. **Make sure that the black line does not come into contact with the urine.**
3. Insert the test into the test cap with the test strips first. **Wait for the reaction time of 5 to 8 minutes. IMPORTANT: The time must be observed exactly to ensure a correct result.** You can find out your personal test result in the Results section.

TEST EVALUATION: The reaction time of this rapid drug test is only 5 to 8 minutes. The test cassette must remain in the cap for the entire time. Then read your result on the test strip. The result must not be evaluated after 10 minutes.

READING PROCEDURE: The test cassette has six windows in the upper, colored part. The drug names that can be evaluated in the test procedure are indicated in these windows. The six reading windows below, marked C and T, are the actual reading windows. A red line must appear on the C line = control line. This indicates that the test is working properly. Your personal test result for each drug is displayed on the T line = test line. If it is positive (i.e. a drug can be detected without any doubt), no line appears at T. Negative test results (i.e. no drug can be detected beyond doubt) are indicated by a reddish line in the reading window.

C = Control line I T = Test line

ACE X	Drug name	OPI	MDA
ACE X	ACE X	MDMA	BAR
ACE X	ACE X	COC	BZD
ACE X	ACE X	AMP	K2
ACE X	ACE X	MET	KET
ACE X	ACE X	THC	TCA


ABBREVIATIONS DRUG NAME

• OPI / Opium - Morphine (e.g. heroin, opiates, codeine,...)	2000 ng/ml
• MDMA / Ecstasy	500 ng/ml
• COC / Cocaine (e.g. cocaine, crack,...)	300 ng/ml
• MTD / Methadone	300 ng/ml
• BAR / Barbiturates (e.g. Secobarbital)	300 ng/ml
• BZD / Benzodiazepines (e.g. Oxazepam)	300 ng/ml
• AMP / Amphetamine (e.g. Speed,...)	1000 ng/ml
• MET / Methamphetamine (e.g. Pep,...)	1000 ng/ml
• K2 / Spice	50 ng/ml
• KET / Ketamine	1000 ng/ml
• TCA / Tricyclic antidepressants	1000 ng/ml
• THC / marijuana, hashish (tetrahydrocannabinol)	50 ng/ml

RESULTS - TEST EVALUATION 1/2


IMPORTANT: The intensity of the C and T lines can vary greatly. **C = Control line:** Proof that the test is meaningful, it has passed.

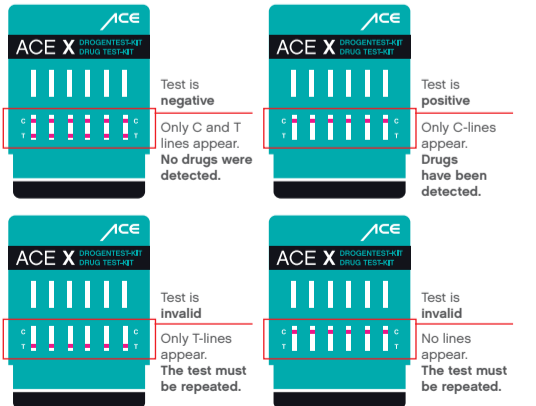T = Test line: Even very faint reddish lines should be evaluated because the quantity of the drug detected is also important (as well as the time of last use, the frequency of use and the concentration/purity of the drug).

RESULTS - TEST EVALUATION 2/2 - Example AMP Amphetamine

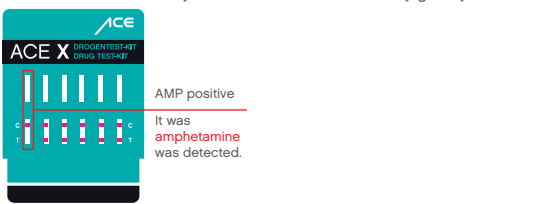IMPORTANT: The intensity of the C- and T-lines can vary greatly

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ACE Handels- & Entwicklung GmbH